




*DVT Prophylaxis for Obese
and Morbidly Obese Patients*

SCD™ Compression System
A-V Impulse System®* Foot Pump
T.E.D.™ Anti-Embolism Stockings

 COVIDIEN

DVT PROPHYLAXIS: SOLUTIONS TO FIT MORE PATIENTS THAN EVER BEFORE.

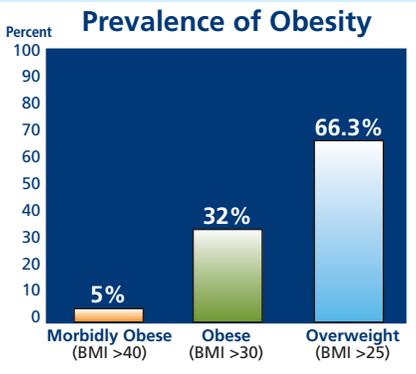

OBESITY is the second most common cause of death in America.¹

In 2006, bariatric **SURGERY** was performed on over 175,000 patients.²

OBESITY and **SURGERY** are independent **RISK FACTORS** for Deep Vein Thrombosis (DVT).³

Size Matters: Obesity increases the Risk of DVT.

Obesity is an ever-increasing problem affecting millions of adults and directly linked to a number of life-threatening diseases. Recent results of the National Health and Nutrition Examination Survey (NHANES) indicate that an estimated 65% of U.S. adults are either overweight or obese, defined as having a body-mass index (BMI) of 25 or more.⁴

- Obese patients have a significantly higher frequency of pulmonary embolism (PE) as the immediate cause of death compared to non-obese patients.⁵
- Studies have shown and suggested leading cause of death is a PE following gastric bypass surgical procedures.⁶
- Overweight and obese patients need a prophylactic DVT modality that fits their unique body shapes.

Obesity and Surgery
Increase the Risk of DVT

Size Matters: Solutions to Fit Obese Patients.

CIRCUMFERENTIAL
SEQUENTIAL
GRADIENT



EXTRA-LARGE KNEE LENGTH

Tailor-made for morbidly obese patients

SCD™ COMPRESSION SLEEVES

The SCD™ compression sleeves have a specific FDA indication for DVT and PE prophylaxis within the at-risk patient population, including the bariatric and morbidly obese patient.

ADVANTAGES

- Clinically proven sequential, gradient, circumferential compression.
- Reinforcing air conduit in the Extra-Large sleeve ensures inflation, even in the presence of the heaviest legs.
- Individual multi-flap closure customizes fit and facilitates fast adjustments.

SIZES:

- Thigh Length sleeves fit thigh circumferences measuring up to 36".
- Knee Length sleeves fit calf circumferences measuring up to 32".

FOOT COMPRESSION



Foot compression is the solution for all sizes. Foot compression is easy to apply and easy to fit, regardless of patient size.

The A-V Impulse System®* foot pump is a dedicated impulse foot compression system backed by decades of clinical studies showing reduction of DVT.

The SCD Express™ compression system is an all-in-one system capable of providing leg compression or slow foot compression.

ADVANTAGES

- Clinically proven impulse compression pattern in the A-V Impulse System®* foot pump.
- Vascular refill technology optimizes compression timing in the SCD Express™ compression system.
- Foot cover shape ensures ease of application and fit.

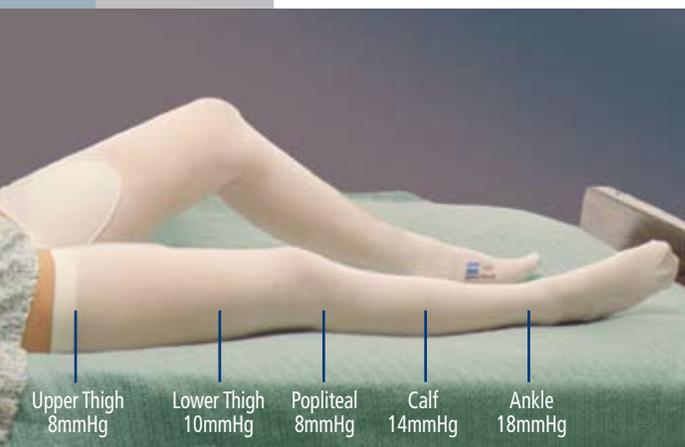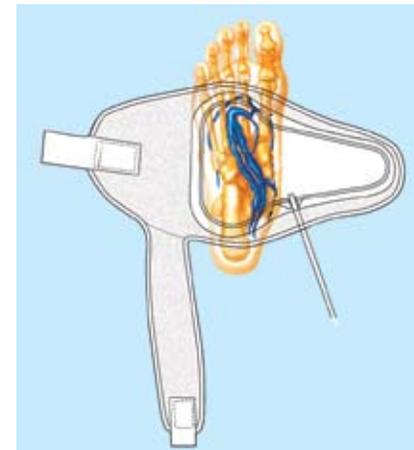

T.E.D.™ ANTI-EMBOLISM STOCKINGS

T.E.D.™ anti-embolism stockings improve patient outcomes when combined with mechanical or pharmaceutical prophylaxis.^{7,8}

ADVANTAGES

- Clinically proven graduated compression pattern increases blood flow and reduces venous distention.
- Provides baseline DVT protection when intermittent compression is interrupted.
- 37 sizes and three styles fit most patients.

SIZES:

- Thigh Length stockings fit thigh circumferences measuring up to 36".
- Knee Length stockings fit calf circumferences measuring up to 26".

SCD™ Compression System

Thigh Length

Thigh Circumference	Size	Item Code		
		Soft Sleeve	SCD EXPRESS	Tear-Away
Up to 22"	Small	5345	9545	9545T
22"—28"	Medium	5330	9530	9530T
28"—36"	Large	5480	9780	9780T
Up to 28"	Sterile	5336	9736	

Knee Length

Calf Circumference	Size	Item Code	
		Soft Sleeve	SCD EXPRESS
Up to 21"	Medium	5329	9529
19"—26"	Large	5489	9789
22"—32"	X-Large	5490	9790

Foot Compression

A-V Impulse System®* Foot Pump

ImPad®* Foot Cover

Size	Women's Shoe Size (U.S.)	Men's Shoe Size (U.S.)	Item Code
Regular	5 ½ – 9	4 ½ – 8	5065
Large	9 ½ – 11 ½	8 ½ – 12 ½	5075
X-Large	13" – 18" in foot circumference		5107

NOTE: For ImPad undercast inflation pads and sterile foot covers, one size fits all.

ImPad™* Under Cast Inflation Pad

Style	Item Code	Style	Item Code
Right foot	5087	Sterile Right foot	6066
Left foot	5089	Sterile Left foot	6067

SCD Express™ Foot Cuffs

Size	Women's Shoe Size (U.S.)	Men's Shoe Size (U.S.)	Item Code
Regular	≤ 9	≤ 7 ½	5897
Large	≥ 9 ½	≥ 8	5898

T.E.D.™ Anti-Embolism Stockings

Knee Length All Stockings are Latex Free

Calf Circumference	Length	Description	Item Code
<12"	<16"	Small Regular	7071
	>16"	Small Long	7339
12" – 15"	<17"	Medium Regular	7115
	>17"	Medium Long	7480
15" – 17.5"	<18"	Large Regular	7203
	>18"	Large Long	7594
17.5" – 20"	<18"	X-Large Regular	7604
	>18"	X-Large Long	7802
20" – 23"	<18"	XX-Large Regular	7470
	>18"	XX-Large Long	7471
23" – 26"	<18"	XXX-Large Regular	7472
	>18"	XXX-Large Long	7473

Thigh Length All X-Large, XX-Large, and LF Suffix Stockings are Latex Free

Thigh Circumference	Calf Circumference	Length	Description	Item Code
<25"	<12"	<29"	Small Short	3071 / 3071LF
		29" – 33"	Small Regular	3130 / 3130LF
		>33"	Small Long	3222 / 3222LF
	12" – 15"	<29"	Medium Short	3310 / 3310LF
		29" – 33"	Medium Regular	3416 / 3416LF
		>33"	Medium Long	3549 / 3549LF
15" – 17.5"	<29"	Large Short	3634 / 3634LF	
	29" – 33"	Large Regular	3728 / 3728LF	
	>33"	Large Long	3856 / 3856LF	
25" – 32"	17.5" – 21.5"	<29"	X-Large Short	3180
		29" – 33"	X-Large Regular	3181
		>33"	X-Large Long	3182
32" – 36"	21.5" – 26"	<29"	XX-Large Short	3183
		29" – 33"	XX-Large Regular	3184
		>33"	XX-Large Long	3185

• SIZES TO FIT THE OBESE PATIENT

Thigh Length with Belt All Stockings are Latex Free

Thigh Circumference	Calf Circumference	Length	Description	Item Code
<25"	<10"	<28"	X-Small Regular	3306
		>28"	X-Small Long	3320
	10" – 12"	<28.5"	Small Regular	3039
		>28.5"	Small Long	3364
	12" – 15"	<28.5"	Medium Regular	3144
		>28.5"	Medium Long	3449
15" – 17.5"	<29"	Large Regular	3221	
	>29"	Large Long	3523	
25" – 32"	15" – 17.5"	<28.5"	X-Large Regular	3922
		>28.5"	X-Large Long	3995

REFERENCES

- Mokdad, A. H., Actual Causes of Death in the United States, 2000. *JAMA*. 2004; 291(10):1238 - 1245.
- Medtech Insight Order #606, "U.S. Surgical Procedure Volumes" *Medtech Insight*. 2007 3-12.
- Geerts, W.H., et al. Prevention of Venous Thromboembolism : The Seventh ACCP Conference on Antithrombotic and Thrombolytic Therapy. *CHEST*. 2004; 126:3385 - 4005.
- National Center for Health Statistics. Health, United States 2004 With Chartbook on Trends in the Health of Americans. Hyattsville, Maryland: 2004 pg 241. <http://www.cdc.gov/nchs/data/has/has04trend.pdf#069>
- MacReady, N. Pulmonary Embolism a Common Cause of Death in the Obese: Presented at ASCP
- Sapala, T.A., et al. Fatal Pulmonary Embolism after Bariatric Operations for Morbid Obesity: A 24-Year Retrospective Analysis. *Obesity Surgery*. 2003; 13:819 - 825.
- Scurr, et al. Regimen for Improved Effectiveness of Intermittent Pneumatic Compression in Deep Venous Thrombosis Prophylaxis. *SURGERY*. 1987; 102 (5): 816-820.
- Torngren, S. Low Dose Heparin and Compression Stockings in the Prevention of Postoperative Deep Venous Thrombosis. *BRITISH JOURNAL OF SURGERY*. 1980; 67:482-484.



®* U.S. registered trademark of Novamedix Distribution Ltd. and is used under license. COVIDIEN, COVIDIEN with logo and ™ marked brands are trademarks of Covidien AG or an affiliate. © 2009 Covidien. All rights reserved.

H-6083 xM xx/xx